

Blood Temp 10

Simple quality assurance for blood temperature during transportation and storage.



Ease of use:

Blood Temp requires no preconditioning or special handling prior to use. Shelf stable, store at room temperature, simple push button activation.


Quality assurance:

Accurately monitors Blood Bag temperature in storage and transit within the Hospital. Alarm status is irreversible.

Compliance:


Blood Temp meets all FDA and AABB requirements: FDA CFR 600. 1 5 (a) for monitoring blood temperature.

FDA 510(k) #BK150289.

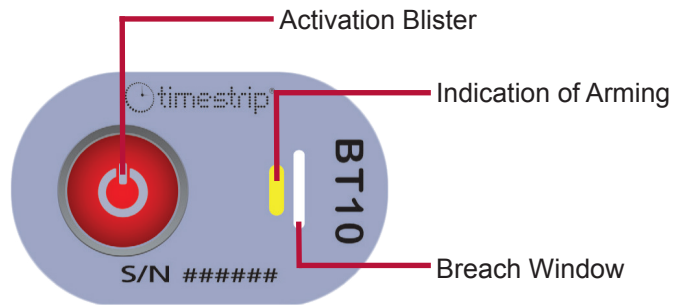


At-a-glance temperature monitoring

Upon activation, Blood Temp 10 instantly monitors blood temperature.

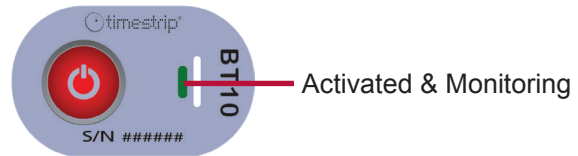

Blood Temp 10 clearly indicates when temperature breaches 10°C. Color change is irreversible and cannot be reset or changed.

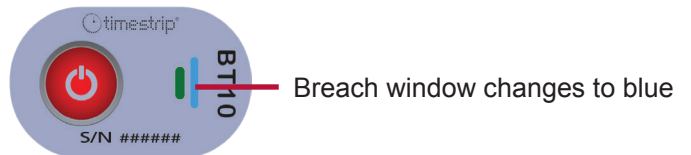

1 Push the Button

Firmly squeeze the activation button until you feel a pop, then release.

2 Verify Activation

Arming window will change from yellow to dark green immediately. Dark green means activated. If not, go back to step 1 and repeat.

3 Apply to Blood Bag

Remove adhesive liner and adhere activated indicator to the lower third of the blood bag.

Key Specifications	
Monitored Temperature	10°C / 50°F
Temperature Accuracy	± 0.5°C ± 1°F
Storage Conditions	Store in a cool, dark place. Recommended 22°C / 72°F at 20-80% RH
Shelf Life	1 year from date of manufacture
Size	1.25in x 0.75in 32mm x 19mm